# Leading cuff technology. It simply runs in our blood.

CRITIKON® blood pressure cuffs by GE Medical Systems




#### **Contents**

Philosophy	3
Standardisation	4
Infection Control	6
Sizing	8
DURA-CUF®	10
SOFT-CUF®	12
CLASSIC-CUF®	14
SENSA-CUF™	16
CRITIKON® Neonatal Cuffs	18





#### The total solution

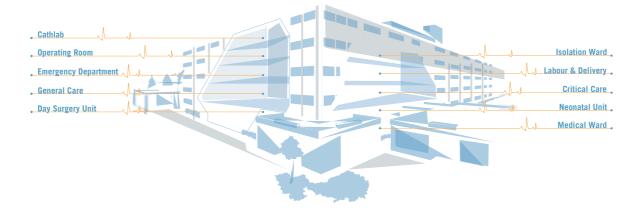



#### CRITIKON®, a leading brand

At GE we believe that satisfying our customers' needs is worth every effort. These efforts lead to solutions which are reliable, helpful and user-friendly.

The state-of-the-art technology used in the sophisticated manufacturing process ensures that CRITIKON® blood pressure cuffs always meet highest quality standards. Four different product families provide blood pressure cuffs for all clinical needs. No matter which application, which size, which monitoring device: GE has the solution.



## With our help, you can spend your time with your patients. And with nothing else.

In a daily hospital routine every minute is precious.

Which is why it is surprising to see that finding the right cuff for the right monitor can be quite complicated. Not so with the CRITIKON® line. Selecting, using, connecting and ordering is just as simple as measuring itself. To be compatible with any blood pressure monitor in your hospital, CRITIKON® blood pressure cuffs come with specially designed connectors which fit to all major monitors.

simple selecting, using, connecting and ordering

connector system for blood pressure monitors

#### **Standardisation**

#### Improving efficiency with CRITIKON®

universal connector system with X and Y adaptors

> for all major types of monitors

But it can even be easier than that: the magic word is standardisation.

CRITIKON® has developed an intelligent way of using one blood pressure cuff for any monitor in your hospital: the universal connector system. With the use of X and Y adaptors CRITIKON® two-tube blood pressure cuffs can be connected to all major types of NIBP monitors. This means maximum convenience for medical and nursing staff, who will have the correct blood pressure cuff available at all times and anywhere in the hospital.



fewer purchase orders, lower stocks, one supplier, financial savings

sales representative support

The further benefit of CRITIKON® lies in lower stocks, fewer purchase orders, one supplier and - consequently - significant financial savings. Could you expect more than that? We believe you could: Call your local CRITIKON® sales representative, he or she will be there to help you realize cost savings in the area of blood pressure measurement.

**GE** can provide the X or Y adaptors to complete your standardisation:















#### **Infection Control**

### The CRITIKON® concept against cross-infection

In hospitals all over the world, people are fighting the danger of cross-infection in order to help and heal patients.

But in today's healthcare systems cross-infection is also a financial risk, which may jeopardize efficient medical care.

financial risk of cross-infection

GE's cuffs for single patient use are an efficient instrument in the fight against cross-infection. We offer a full range of single patient use of cost-efficient products for single patient use anywhere in the hospital where there is a danger of cross-infection. The safest way of minimizing the risk.

#### **Infection Control**

#### Shared responsibilities

best cleaning results
with no anti-microbial
agents



For departments where you prefer to use multi-patient cuffs, CRITIKON® products come with a durable and easy-to-clean surface. This guarantees best cleaning results without having to use antimicrobial agents.

Avoiding the use of anti-microbial agents is an integral part of the CRITIKON® cuffs philosophy. We know that the use of these agents adds to the development of resistant microbes. That is why we believe that we have the responsibility to create high hygiene standards with other, more responsible concepts.

For optimum results, CRITIKON® blood pressure cuffs come with detailed cleaning and disinfection instructions.

detailed cleaning and disinfection instructions

- o CONCLUSION: This study emphasizes the need for increased awareness of the potential for cross-contamination to patients and health care workers from seemingly innocuous items of general use hospital equipment, such as blood pressure cuffs.

  (THE MICRONIAL FLORA OF IN-USE BPC by M.G.M. Cormican, D.L. Lowe, P. Flynn, D. O'TOOLE Irish Journal of Medical Sciences 1994; 4; 112-3)
- "...It is underappreciated that all major groups of microorganisms - viruses, fungi, and parasites as well as bacteria - become resistant to antimicrobials." (Anthony S. Fauci, M.D., Director, National Institute of Allergy and Infectious Diseases, National Institutes of Health, USA, 1999)

#### Sizing

range of cuffs with various sizes


Choosing the correct size is important for accurate blood pressure measurement and maximum patient comfort. This is why CRITIKON® has developed a simple and easy cuff sizing system to cater for all patients' needs.

The full range of sizes goes from limb circumference of 3 cm to 50 cm for the thigh cuff.

easy-to-use arterial markers and index lines

colour-coded for size selection

The functional design of CRITIKON® cuffs includes easy-to-use arterial markers and index lines for proper size and placement. Additionally, our cuffs are colour-coded by size to make size selection as easy as possible.

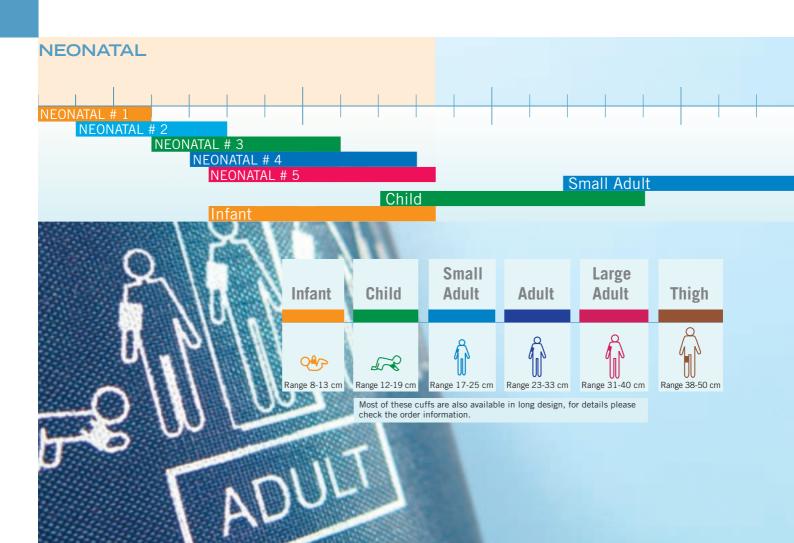

#### Sizing

meet ANSI/AAMI SP10 and EN1060 standards


CRITIKON® blood pressure cuffs meet the recommendations of the American Heart Association as well as ANSI/AAMI SP10 and EN1060 standards.

In response to the increasing amount of allergic reactions against latex amongst health workers all CRITIKON® blood pressure cuffs are latex-free.

latex-free

CONCLUSION: The researcher points out that several surveys have proven, that undersized cuffs overestimate and oversized cuffs underestimate the true BP by as much as 10 to 30 mmHg. Under cuffing can lead to the over treatment of hypertension and over cuffing to its under treatment...

(MISCUFFING: INAPPROPRIATE BPC APPLICATION by D.M. Manning, C. Kuchirka, J. Kaminski, Circulation, 1983 Oct; 68(4);763-6)

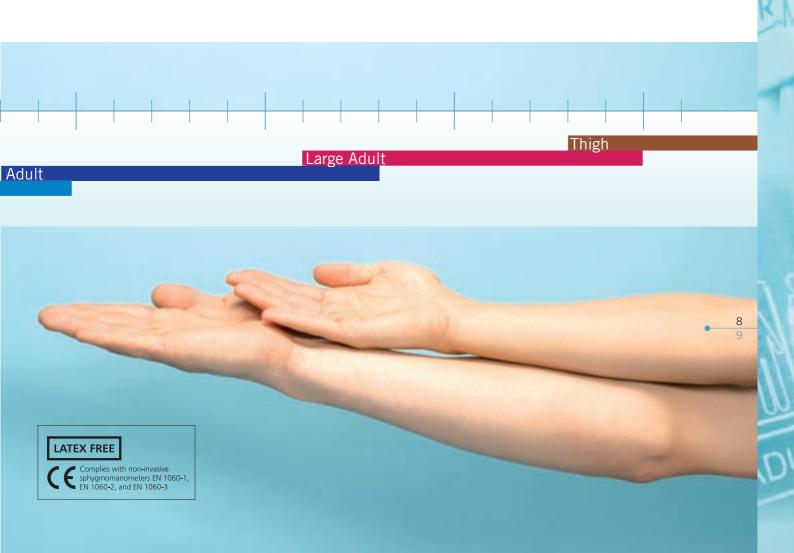

#### **DURA-CUF®**

Gentle nylon, high quality, extra durable

Reusable for multiple patient use

Convenient and easy to clean

Extra strong hook and loop

Arterial markers and index line for accurate fit

Latex- and PVC-free •



DURA-CUF® is the ideal all-round blood pressure cuff for the demanding day-to-day work almost anywhere in a hospital. It is designed as a multi-patient blood pressure cuff for areas with a low risk of infection.



#### **DURA-CUF®**





DURA-CU	IF® Assortment Packs - 2-Tube Screw Connectors	DURA-C	UF® Assortment Packs - 2-Tube Submin Connectors
002699	1 Infant, 1 Child, 1 Small Adult, 1 Adult, 1 Large Adult, 1 Thigh	2299	1 Infant, 1 Child, 1 Small Adult, 1 Adult, 1 Large Adult, 1 Thigh
002697	Child Assortment: 2 Infant, 3 Child, 1 Small Adult	2296	Child Assortment: 2 Infant, 3 Child, 1 Small Adult
002698	Adult Assortment: 1 Small Adult, 2 Adult, 1 Adult Long, 1 Large Adult, 1 Large Adult Long	2297	Adult Assortment: 1 Small Adult, 2 Adult, 1 Adult Long, 1 Large Adult, 1 Large Adult Long

#### SOFT-CUF®

Soft, absorbent material

**Economical for single patient use** 

Colour-coded by size

Arterial markers and index line for accurate fit

Latex-free In terms of patient comfort and infection control SOFT-CUF® is the right choice.

> SOFT-CUF® is an excellent choice for all clinical areas with increased infection risk. Using the SOFT-CUF® as a single patient use cuff reduces the care staff's work load and improves the workflow.



#### SOFT-CUF®





**SOFT-CUF®** Assortment Packs - 2-Tube Screw Connectors

**SOFT-CUF® Assortment Packs - 2-Tube Submin Connectors** 

**002695** 1 Infant, 1 Child, 2 Small Adult, 2 Adult, 1 Adult Long, 2 Large Adult, 1 Thigh

2298 1 Infant, 1 Child, 2 Small Adult, 2 Adult, 1 Adult Long, 2 Large Adult, 1 Thigh

#### **CLASSIC-CUF®**

Easy-to-clean vinyl surface

Strong enough for repeated inflations

Arterial markers and index line for accurate fit

Available in a wide range of sizes from neonate to adult

Yellow cuff for isolation use

Latex-free



In many operating theatres and intensive care units all over the world, CLASSIC-CUF® has served doctors and nurses with outstanding durability and precise readings.



In clinical settings where the demands on hygiene are high and the infection risk is significant, CLASSIC-CUF® is a reasonable alternative to the use of SOFT-CUF®.


Its excellent surface structure, which is easier to clean than any other blood pressure cuff, is made of vinyl, and is completely latex-free.

The yellow CLASSIC-CUF® is specially designed for isolation use.

# CLASSIC-CUF®

#### **CLASSIC-CUF®**



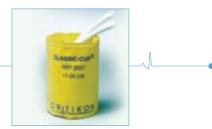

LASSIC- FIEF 2	EUF 603 CM		CRITIKON DINAMAP	PHILIPS/HP SIEMENS SPACELABS DATASCOPE	GE MEDICAL SYSTEMS IT (MARQUETTE	OHMEDA	PROTOCOL	Manual sphyg. Wall Manometer
			1	1	-	1	1	1
Size	Colour	Limb Circum- ference	2-Tube Screw Connector (BX/20)	1-Tube Bayonet Connector (BX/20)	2-Tube Submin Connector (BX/20)	2-Tube Mated Submin Connector (BX/20)	1-Tube Submin Connector (BX/20)	SOFT-CUF® Inflation Systems (BX/5)
Infant	Orange/White	8-13 cm	2618	2300	2354	2170	2150	
Child	■ Green/White	12-19 cm	2613	2301	2355	2171	2151	002287
Small Adult	Lt.Blue/White	17-25 cm	2608	2302	2356	2172	2152	002285
Adult	■ Navy/White	23-33 cm	2603	2303	2357	2173	2153	002283
Adult Long	■ Navy/White	23-33 cm	2647	2306	2352	2174		
Large Adult	■ Rose/White	31-40 cm	2643	2304	2358	2175	2154	002281
Large Adult Long	■ Rose/White	31-40 cm	2649	2307	2353	2176		
Thigh	■ Brown/White	38-50 cm	2648	2305	2359	2177	2155	
For isolation use	:							
Infant	Yellow	8-13 cm	2650		2670			
Child	Yellow	12-19 cm	2651		2671			
Small Adult	Yellow	17-25 cm	2607		2672			
Adult	Yellow	23-33 cm	2602		2673			
Large Adult	Yellow	31-40 cm	2642		2674			

14 15

#### SENSA-CUF®

Soft, breathable and cleanable material


Reusable for multiple patient use

Ideal for extended wear in critical care areas

Arterial markers and index line for accurate fit



Latex and PVC-free • SENSA-CUF® is possibly the most flexible blood pressure cuff on the market. Designed to offer all important benefits of the CRITIKON® line - quality, various sizes and best functionality - it is suitable for multi-patient use. So it closes the gap between the long-lasting DURA-CUF® and single-patient SOFT-CUF®.

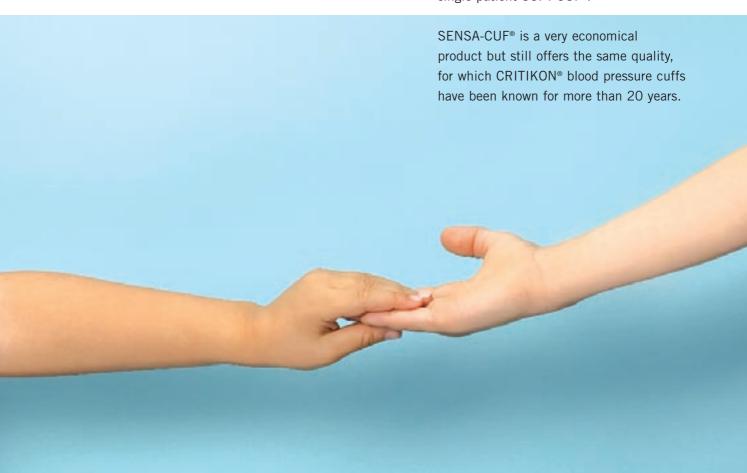

#### SENSA-CUF®





SENSA-CUF® Assortment Packs - 2-Tube Screw Connectors

2494 1 Small Adult, 1 Adult, 1 Large Adult

#### **CRITIKON® Neonatal Cuffs**

Choice of soft absorbent material or traditional vinyl

Single patient use

Playful animal designs help to find the right size at a glance

Arterial markers and index line for accurate fit

Use with neonatal air hoses

Latex-free



Because the youngest patients often need the most careful treatment, GE offers two ranges of CRITIKON® blood pressure cuffs for premature infants and neonates.




SOFT-CUF® has the softest and most absorbent surface, giving small patients the comfortable monitoring they deserve. Used within the single-patient regimen, SOFT-CUF® efficiently helps to avoid nosocomial infections.

Another option for the care of premature infants and newborns in critical areas of the hospitals is the neonatal CLASSIC-CUF®.

SOFT-CUF® and CLASSIC-CUF® are both available in 5 neonatal sizes, and their decorative designs help to identify the correct size at a glance.

#### **CRITIKON® Neonatal Cuffs**



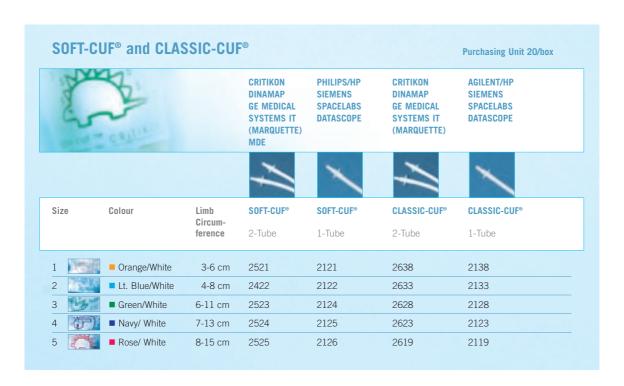

SOFT-CI	UF® Assortment Packs - 2-Tube Luer Connectors	SOFT-CU	IF® Assortment Packs - 1-Tube Luer Connectors
2694	2 Neo #1, 3 Neo #2, 5 Neo #3, 5 Neo #4, 5 Neo #5	2696	2 Neo #1, 3 Neo #2, 5 Neo #3, 5 Neo #4, 5 Neo #5
CLASSI	C-CUF® Assortment Packs - 2-Tube Luer Connectors	CLASSIC	C-CUF® Assortment Packs - 1-Tube Luer Connectors

For more than 100 years, healthcare providers worldwide have relied on GE Medical Systems for medical technology, services and productivity solutions.

So no matter what challenges your healthcare system faces - you can always count on GE to help you deliver the highest quality healthcare.

For details, please contact your GE representative today.



#### **GE Medical Systems** Information Technologies

European Headquarters
GE Medical Systems Information Technologies GmbH

P.O. Box 60 02 65 • 79032 Freiburg, Germany Tel. +49 761 45 43 - 0 • Fax +49 761 45 43 - 233

#### World Headquarters

GE Medical Systems Information Technologies, Inc. 8200 West Tower Avenue • Milwaukee, WI 53223, USA Tel. +1 414 355 5000 • Fax +1 414 355 3790

#### Asia Pacific

GE Medical Systems Information Technologies Asia 24F Floor, Shanghai MAXDO Center, No. 8 Xing Yi Road, Hong Qiao Development Zone • Shanghai 200336, P.R. China Tel. +86 21 5257 4650 • Fax +86 21 5208 2008

General Electric Company reserves the right to make changes in specifications and features shown herein, or discontinue the product described at any time without notice or obligation. Contact your GE representative for the most current information.

© 2003 General Electric Company